MICROLINE™ SURGICAL

Microline Surgical, Inc.
50 Dunham Road, Suite 1500
Beverly, MA 01915 U.S.A.
Tel: +1(978) 922-9810 | Fax: +1(978) 922-9209
Web: www.microlinesurgical.com
Email: info@microlinesurgical.com

09-39-30001-00-ART REV J

ENGLISH

Visu-Loc 5mm Multi-Fire Spring Clip Applier, 20 Clips, Disposable Visu-Loc 3mm Multi-Fire Spring Clip Applier, 10 Clips, Disposable

Instructions for Use

Indications

The Visu-Loc 3mm and 5mm Multi-Fire Spring Clip Appliers are indicated for ligation of tubular structures or vessels in laparoscopic and general surgical procedures.

Contraindications

The Visu-Loc 3mm and 5mm Multi-Fire Spring Clip Appliers are not intended for use except as indicated. They are not intended for use when endoscopic techniques are contraindicated.

Device Description

The Visu-Loc 5mm Multi-Fire Spring Clip Applier has a 5mm metal shaft attached to metal jaws and contains 20 medium/ large titanium clips. The Visu-Loc 3mm Multi-Fire Spring Clip Applier has a 3mm metal shaft attached to metal jaws and contains 10 medium/ large titanium clips. The handle and trigger are plastic. The Visu-Loc is designed for introduction and use through a 3mm or 5mm trocar cannula port or incision.

Instructions For Use

To insert the Visu-Loc through a trocar cannula or incision, squeeze the trigger just until the jaws close. After insertion through the trocar cannula or incision, release the trigger and the jaws will reopen. By turning the rotator, adjust the jaws for maximum visibility. To apply a clip, place the tissue to be ligated against the back of the open jaws. Squeeze the trigger completely, so the jaws close around the tissue, deploying the Visu-Loc clip. A clicking sound will be heard and felt. Releasing the trigger completely will allow the reposition and application of another clip. To remove the Visu-Loc from the trocar cannula, squeeze the trigger just until the jaws close. Remove the Visu-Loc and release the trigger. The Visu-Loc is ready for further use, or it may be discarded.

Warnings and Precautions

- Procedures for endoscopic surgery should be performed only by qualified and trained physicians familiar with endoscopic surgical techniques.
- A thorough understanding of the principles and techniques involved in laparoscopic and endoscopic surgical procedures is essential to avoid shock and burn hazards to both patient and medical personnel and damage to the device or other medical instruments.
- When endoscopic instruments and accessories from different manufacturers are employed together in a procedure, verify compatibility prior to initiation of the procedure and ensure that electrical insulation or grounding is not compromised.
- Attempting to penetrate or twist tissue with the jaws in the open position can result in damage to the jaws and render the 3mm or 5mm clip applier inoperable. Close the jaws when pressing hard or twisting through tissue.
- Ensure the tissue or vessel fits completely within the confines of the clip's legs or ligation and/or hemostasis may be compromised.
- When dividing the tissue next to the clip, the length of remaining cuff must be equal to, or greater than, the diameter of the vessel. Dividing the tissue immediately next to the clip, or otherwise leaving a short inadequate cuff, may result in the clip slipping off the tissue, compromising hemostasis.
- This device is not intended for contraceptive tubal occlusion but may be used to achieve hemostasis following transection of the fallopian tube.
- Do not use the 3mm or 5mm clip applier on vessels upon which metal ligating clips would not normally be used.
- Inspect the ligation site to ensure proper application and that hemostasis has been achieved.
- The Visu-Loc is sterilized by EO. It is a sterile, single use product and cannot be resterilized.
- Resterilization may compromise the integrity of the product which may result in unintended injury. Discard a device with compromised packaging.
- Data indicates that the physical characteristics and quality of this device will be adversely affected and that the device will not remain safe and effective for its intended use when cleaned and resterilized.

WARRANTY

Microline Surgical warrants that its instruments are free from any defects in both material and workmanship. Microline Surgical shall not be held liable for any incidental or consequential damage of any kind. Work performed on an instrument will void this warranty. Gross abuse or neglect of a Microline Surgical instrument will void this warranty.

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

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