MICROLINETM SURGICAL

ReNew Family of Laparoscopic Hand Pieces

3900 Series Ratcheted and Non-Ratcheted

Instructions For Use

Microline Surgical, Inc.
50 Dunham Road
Beverly, MA 01915, U.S.A.
TEL: (978) 922-9810
FAX: (978) 922-9209

Web: www.microlinesurgical.com
Email: info@microlinesurgical.com

09-39-36 Rev. F. 10/2014

MICROLINETM SURGICAL

ENGLISH

ReNew Family of Laparoscopic Hand Pieces 3900 Series Ratcheted and Non-Ratcheted

INDICATIONS
The ReNew Hand Piece and tips are indicated for cutting, grasping, dissecting and coagulation of tissue in endoscopic and laparoscopic surgical procedures.

CONTRAINDICATIONS


The ReNew Handpiece and tips are not intended for use except as indicated.

PRIOR TO USE

Carefully inspect the shipping carton and its contents for any sign of damage. If damage is visible, do not use the instrument. This instrument is delivered without a tip. Tips are available in various configurations, (see catalog).

ERGONOMICS

This instrument is delivered with two sets of separately packaged reusable nonlatex rings (small & medium sizes). These rings are designed to be mounted around the inside of both handle finger holes and they are intended to maximize surgeon comfort by adjusting the inside size of the hand piece finger rings. Depending on what the surgeon feels most comfortable with, the small, medium, or no ring sets can be used during a procedure.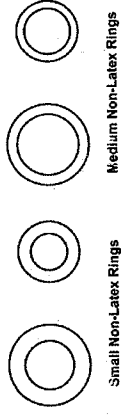

HAND PIECE / TIP ASSEMBLY

- Gently close the blades/jaws with your index finger and your thumb to ensure complete closure (See Fig. 2).
- With blades/jaws of tip and handles of hand piece in their closed position, insert the tip into the shaft.
- Hold the jaws and turn the hand piece rotation knob clockwise until tip is screwed tight.
- The instrument is ready for use

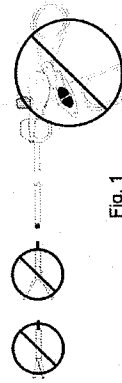
Fig. 1

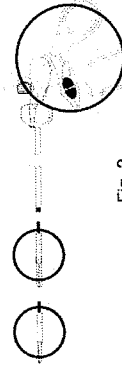
Fig. 2

OPERATING INSTRUCTIONS FOR RATCHETED HAND PIECES

- Push switch (see Fig. 3) forward in order to engage the ratchet mechanism.
- Close the handles to the desired grasping position and let go. The instrument is locked on the tissue.
- To release the grasper jaws, press down on the locking lever.
- In order to use the instrument as a non-ratcheted one, pull the switch toward the back of the instrument. The instrument will open and close freely.

NOTE

Non-Ratcheted mode is necessary to screw a tip on to the hand piece.

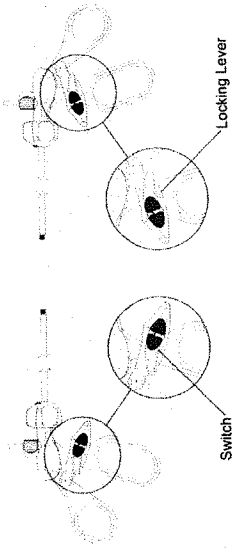
Fig. 3

PRECAUTIONS

- If the blades/jaws are not completely closed when screwed onto the hand piece, they may not cut or grasp completely at the distal tip.
- CAUTION: Prior to use, assure that the plastic hub of tip is fully in contact with the insulation tube of hand piece and that no gap exists between the two parts at that contact point.
- Do not use hand piece if the "O" ring located on the distal end of the shaft appears worn, damaged or missing.

ELECTROCAUTERY

First, connect the electrosurgical cord (not furnished with the instrument) to the hand piece by placing the female end of the cord on the male adapter pin on the hand piece. Plug the other end of the cord into the Monopolar receptacle of the generator.

PRECAUTIONS

- A complete understanding of the principle of monopolar electrocautery surgical procedures is necessary to avoid accidental shocks, burns, or potential gas embolism to the patient.

- Be sure that a return electrode grounding pad has been properly attached to the patient and the generator.
- Do not activate the generator until the scissors' blades are in contact with the tissue or are in a position to deliver high frequency energy to the tissue.
- Keep the power as low as possible to achieve the desired effect.

Note: The ReNew hand piece has an O-ring, at the distal end of the shaft, which is an integral part of the system. Prior to each use, it is very important to verify that the O-ring is in place and it is not damaged. If the O-ring is missing or damaged, do not use the instrument.

CLEANING

Note: Microline SURGICAL recommends that tips and non-latex rings be detached from hand piece before cleaning and sterilization.

- Presoak the instrument in enzymatic solution containing 60mL of ENZOL® Enzymatic Detergent per 4 L of water @ 22-40°C.
- Remove (unscrew) the cautery plug.
- Insert a 60 ml luer type syringe filled with warm tap water where the cautery plug was removed. Flush the inside of the shaft and tip clean.
- Brush the distal (working end) of the instrument under water, to avoid airborne contaminants.
- Make sure that all visible bioburden has been removed under water.
- Flush inside of the shaft three (3) times with deionized water for a final rinse.
- Inspect the instrument for functionality and package appropriately for sterilization.

STERILIZATION

- Prior to sterilization, the instrument must be thoroughly cleaned.
- Wrap the instrument.

Microline SURGICAL recommends using the following validated steam sterilization cycles as guidelines:

Gravity Cycle: 4 minutes @ 270°F (132°C)
Gravity Cycle: 30 minutes @ 250°F (121°C)
Pre-Vacuum Cycle: 4 minutes @ 270°F (132°C)

STERILIZATION PARAMETERS FOR EUROPEAN COUNTRIES

Microline SURGICAL recommends the following parameters as minimum sterilization cycle parameters for Europe, except for France and Switzerland:

Pre-Vacuum Cycle: 3-5 minutes at 134°C

For France and Switzerland, the following minimum sterilization cycle parameters are recommended:
Pre-Vacuum Cycle: 18 minutes at 134°C, 2x102 kPa.

WARRANTY INFORMATION

Microline Surgical warrants that its instruments are free from any defects in both material and workmanship. Microline Surgical shall not be held liable for any incidental or consequential damaged of any kind. Work performed on an instrument will void this warranty. Gross abuse or neglect of a Microline Surgical instrument will void this warranty.

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

CONTACT INFORMATION
Microline Surgical, Inc.
50 Dunham Road
Beverly, MA 01915, U.S.A.
TEL: (978) 922-9810
FAX: (978) 922-9209

Medical Device Safety Service GmbH
D-30175 Hannover
Germany

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