Microline Surgical, Inc.  
50 Dunham Road  
Beverly, MA 01915 U.S.A.  
TEL: (978)922-9810

## PowerPack<sup>TM</sup> Dual Control Footswitch – Instructions For Use

For use only with

Microline Cautery Instruments, Microline PowerPack<sup>TM</sup> Universal Power Supply (UPS) and Microline UPS (REF 200-006R, 200-004R)

### Device Description

The PowerPack Dual Control Footswitch is a non-sterile, reusable unit intended to be used only with Microline Cautery Instruments and the Microline UPS. The PowerPack Dual Control Footswitch has one outlet cable which is connected to the UPS. The PowerPack Dual Control Footswitch has two power settings, variable and high. The amount of power delivered in the variable mode is determined by the knob/push button setting on the power supply. Variable power is activated by depressing the left footpedal. High power is activated by pressing on the right footpedal (see figure). The PowerPack Dual Control Footswitch should be placed on the floor or a flat surface (such as a stand) near the surgeon so that it can be activated by the surgeon's foot.


To use the different heat ranges, the operator must depress the appropriate foot pedal on the PowerPack Footswitch while activating the safety interlock of the device. Generally, lower heat ranges increase the sealing capabilities and increase the time required to divide tissue. Higher heating ranges decrease the time to divide and may decrease seal integrity. See appropriate device instructions for use.

### Intended Use

The PowerPack Dual Control Footswitch is intended for use only with Microline Cautery Instruments and the UPS for the simultaneous cutting and coagulation of soft tissue during surgery.

1. Connect the PowerPack Dual Control Footswitch to the non-sterile UPS by inserting the connector of the PowerPack Dual Control Footswitch into the footswitch connector receptacle of the power supply. Ensure connector plug keys properly into receptacle.
2. Insert the connector of the Cautery Instrument power cord into the instrument connector receptacle (non-sterile) of the UPS. Ensure connector plug keys properly into receptacle.
3. Place the PowerPack Footswitch on a level surface near enough to the surgeon so that it can be activated by the surgeon's foot.
4. Refer to the appropriate Cautery Instrument and Microline UPS Instructions for Use for device-specific use instructions.
5. When using footswitch activated devices, depressing the PowerPack Dual Control Footswitch pedal will activate the heating element only after closing cautery device (see device specific instructions for use).
6. Activate the Cautery Instrument by first closing the device and then depressing the left or right footpedal (variable or high power respectively), depending on the desired clinical result. When the heating element of the Cautery Instrument is activated, the UPS emits an audible tone that lasts until the instrument heating element is deactivated. A low pitch tone is emitted when variable power is delivered. A high pitch tone is emitted when high power is delivered.
7. At the conclusion of the surgical procedure, disconnect the PowerPack Dual Control Footswitch from the UPS.

Note: Unlike the Cautery Instrument, which is discarded at the end of the surgical procedure, the PowerPack Dual Control Footswitch is reusable and should be moved to a safe and convenient storage location for use in future surgical procedures.

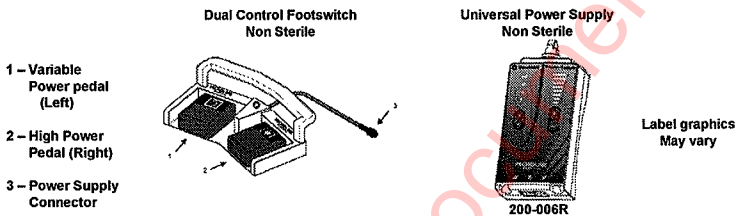### Precautions And Warnings

- Use only with Microline Cautery Instruments and UPS. Use of any other instrument/power supply may damage the instrument, the power supply and/or the PowerPack Dual Control Footswitch and could prevent it from functioning properly during use.
- Do not drop the PowerPack Dual Control Footswitch.
- Do not immerse the PowerPack Dual Control Footswitch in liquids.
- Do not sterilize PowerPack Dual Control Footswitch.
- Procedures using instruments for cutting/cauterization should be performed only by persons having adequate training and familiarity with these surgical techniques. Consult the medical literature relative to techniques, complications and hazards prior to performance of any procedure. Surgeons using this device should be familiar with the specific anatomy of the region in which they intend to perform the procedure.
- There are no user-serviceable parts.

### Warning:

Do not use in the presence of flammable materials (e.g. alcohol, flammable anesthetics).

Always disconnect the instrument before discarding; the PowerPack Dual Control Footswitch and UPS are reusable.

### Handling, Storage And Cleaning

Do not drop. Store in a cool, dry place. Avoid prolonged exposure to extreme temperatures. Exterior of the PowerPack Dual Control Footswitch may be cleaned using a soft cloth moistened with a solution of water and mild detergent or disinfectant.

Do not immerse the PowerPack Dual Control Footswitch in liquids.

### Limited Warranty

Microline Surgical warrants that its instruments are free from any defects in both material and workmanship. Microline Surgical shall not be held liable for any incidental or consequential damage of any kind. Work performed on an instrument will void this warranty. Gross abuse or neglect of a Microline Surgical instrument will void this warranty.

### Symbol Definition

Means: Variable Power.

Means: High Power

Means: Does not Contain Latex.

Medical Device Safety Service GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany

IFU0025 Rev G

Manufactured under one or more of US patents: 7,033,351; 6,908,463; 6,860,880; 6,695,837; 6,626,901. Patents Pending. All Rights Reserved.

© Copyright 2011, Microline Surgical, Inc. All rights reserved.

Microline Surgical, Inc.  
50 Dunham Road  
Beverly, MA 01915 U.S.A.  
TEL: (978)922-9810

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© Copyright 2011, Microline Surgical, Inc. Tous droits réservés.
© Copyright 2011, Microline Surgical, Inc. Tutti i diritti riservati.
© Copyright 2011, Microline Surgical, Inc. Alle Rechte vorbehalten.
© Copyright 2011, Microline Surgical, Inc. Reservados todos los derechos.
© Copyright 2011, Microline Surgical, Inc. Alle rechten voorbehouden.
© Copyright 2011, Microline Surgical, Inc. Todos os direitos reservados.